

Cutting Guide

Ears



Comfeel[®] Plus Transparent to protect skin at risk of pressure injuries on areas where masks, goggles, or oxygen tubing rubs on the ears.

Comfeel[®] Plus Transparent (5x7cm)



- Cut to approx 2cm width.
- Then cut to shape as shown.
- Anchor to the flat of the head with the uncut half.
- Put the other two sections around the ear.


Other options with Comfeel[®] Plus Transparent & Biatain[®] Non-Adhesive for wounds, or to protect areas on the ears.

Comfeel[®] Plus Transparent (5x7cm)



- Cut into a strip, fold in half lengthways, then cut a 'fringe' along one edge (approx. 0.5 cm apart).
- Apply the uncut edge of the dressing first.
- Use the cut edges to fold around the shape of the ear.
- If the wound is wet, use a small piece of alginate underneath.

Biatain[®] Non-Adhesive (5x7cm)

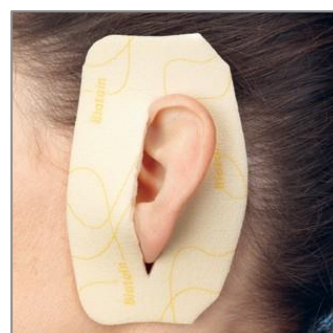

- Cut the dressing in order to fix the size of the ear.
- Secure with a suitable secondary dressing.